# FeetMe<sup>®</sup> Evaluation & FeetMe<sup>®</sup> Monitor

GB

FR

## Instruction for use

---

**GB**

English

## Table of content

1. Labels and symbols .....	5	a. First use or reinstallation b. Perform a gait recording c. View the data acquired from the mobile application d. Backup data e. View battery level f. Update the application	
2. Product scope .....	6		
3. User profile .....	6		
4. About this manual .....	6		
5. Introduction .....	7		
6. Important information and safety instructions .....	7		
7. Product description .....	9		
8. Operating principle .....	9		
9. Product setup .....	9		
a & b. Charging the insoles with a unit charger & the charging case c. Charge light indicators d. Bluetooth connection to the insoles			
10. Use of insoles with FeetMe® Evaluation mobile application .....	11		
		11. Troubleshooting and deactivation .....	12
		12. Storage and transport .....	13
		13. Cleaning and maintenance .....	13
		14. Disposal .....	13
		15. Technical characteristics .....	14
		16. Recommended minimum phone features .....	15
		17. Characteristics of the radio module .....	15
		18. Maintenance .....	17
		19. Lifetime .....	17

## 1. Labels and symbols

	Name and address of the manufacturer		Keep it dry
	Date of manufacture		Moisture limit for storage and transport
	Limiting temperatures for storage and transport		Limiting pressure for storage and transport
	Serial number		Type BF Applied Parts (this device is protected from leakage currents and electric shocks).
	Important safety instructions or operating instructions		Electromagnetic non-ionizing radiation. This equipment includes RF transmitters. Interference may occur around the equipment that has this symbol.
	Double insulation		This device uses Bluetooth
	Refer to the instruction manual / brochure		Medical device in compliance with the directive 93/42 / EEC established by notified body No 0459
	Electronic equipment subject to selective collection		

## 2. Product scope

### **Warning**

This device is a medical device consisting of connected insoles (FeetMe® Monitor) that allow the collection of the gait parameters and a mobile application (FeetMe® Evaluation) that allows visualization in real time of these parameters.

### 3. User profile

There are two types of users:

- Health professionals who deliver insoles to their patients, record and analyze the data.
- Patients wearing insoles during walking tests.

Referring health professional prerequisite : health professional acknowledges having read and fully understood the information on uncertainty of measurement (section 15 page 13 Technical Specifications). For any further information concerning interpretations of uncertainty of measurement of the device, contact FeetMe at [contact@feetme.fr](mailto:contact@feetme.fr)

	Characteristics
Weight	< 135kg
Sex	W/M
Health conditions	All
In contact with	Foot
Age	> 12 years old (sizes from 35 and at least 8 years of education)
Required Skills	Use of a mobile application Use of a website

### 4. About this manual

We recommend that you read this entire manual carefully before starting to use the device. This manual will provide you with the necessary information for a proper use and maintenance of the device. For more information, please contact FeetMe by email at : [contact@feetme.fr](mailto:contact@feetme.fr)

### **Warning**

Be careful, do not use the device under any conditions other than those specified in this manual.

### 5. Introduction

FeetMe device, intended for healthcare professionals and patients, makes it possible to represent walking parameters in order to perform a biomechanical analysis. The medical device comprises :

FeetMe® Monitor insoles for measuring plantar pressure and spatio-temporal parameters of walking.

The FeetMe® Evaluation mobile application which allows the visualization of parameters and the recording of these parameters.

### 6. Safety instructions

Both the installation and use of FeetMe® Evaluation do not require additional training. However, the following points must be respected :

- FeetMe® Monitor insoles are not intended to be in contact with bare skin nor wounds. They should be worn with socks.
- Make sure that FeetMe® Monitor insoles are charged (please refer to the corresponding section of this manual).
- Do not use FeetMe® Monitor insoles while they are charging.
- Always plug your device into a wall outlet.
- In the event of a defect (please refer to this manual for the identification of potential defects), do not use the device and notify the manufacturer or distributor for a repair or replacement.
- Do not drop FeetMe® Monitor insoles or let them be subject to a severe shock.
- Do not place the power cable around your neck.
- Do not use this device in conjunction with other Bluetooth devices.
- All mobile app updates should be accepted.
- FeetMe® Evaluation device requires special precautions in regards to electromagnetic compatibility. The device must be installed and operated according to the electromagnetic compatibility information provided in this manual.
- Portable and mobile Radio Frequency communication devices may affect FeetMe® Evaluation device.

- Be careful not to puncture or cut the soles and their battery and not to access the electronic reading circuit directly.
- Do not position the wall adapter or charging cable in a position where the unplugging is made difficult to the user.
- Performance that could be lost or hindered by FeetMe® Evaluation device due to electromagnetic interference : none.
- Appliances such as microwaves can compromise the operation of the insoles. Be careful not to use your insoles near this type of devices.
- Do not touch the magnetic elements of the insole directly.
- Change shoes in order to use FeetMe® Evaluation insoles in case they are not comfortable or fit your footwear.
- FeetMe® Evaluation insoles do not match all foot shapes.
- No special precaution is necessary in environments of use.

**Warning**

The device must be charged with the wall adapter reference specified on page 13 (Technical characteristics). The use of a wall adapter other than the specified one poses the risk of an electric shock.

**Warning**

This device should not be used near other devices or stacked above other devices as this may cause malfunction. If this use is necessary, this unit and other devices should be observed to ensure their normal operation.

**Warning**

RF portable communications devices (including peripherals such as antenna cables and external antennas) should not be used closer than 30 cm (12 inches) to any parts of the FeetMe® Monitor insoles, including cables specified by the manufacturer. Otherwise, the performance of these devices may be impaired.

**Warning**

Use of accessories, transducers, and cables other than those specified or provided by the manufacturer may cause increased electromagnetic emissions or decreased immunity of this equipment and therefore cause improper operation.

**Warning**

Any modification made to FeetMe® Monitor insoles and FeetMe® Evaluation application is forbidden.

**7. Product description**

The device consists of a pair of connected insoles and an Android application (FeetMe® Evaluation) running on a smartphone.

The mobile application is downloadable on smartphone.

Detachable parts
Double head charge cable (induction charger)
Usb wall adapter (see §15)

**8. Operating principle**


FeetMe® Monitor insoles are powered by a battery and measure pressure on the plantar surface alongside walking parameters. These data are calculated locally with in the insole and transmitted by Bluetooth to the smartphone.

The FeetMe® Evaluation app can be used on a smartphone to receive, store and view the calculated gait parameters.

**9. Product setup**

Setup does not require any tool other than those provided with the device.

**a. Charging the insoles with a unit charger**

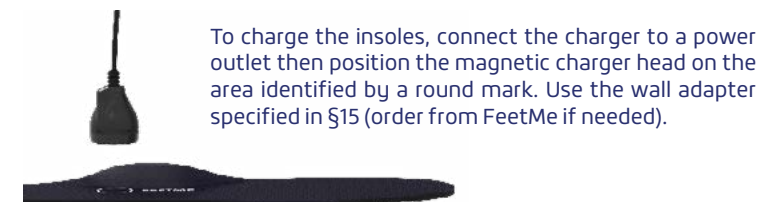To charge the insoles, connect the charger to a power outlet then position the magnetic charger head on the area identified by a round mark. Use the wall adapter specified in §15 (order from FeetMe if needed).

## b. Charging the insoles with the charging case

### Setting up the charging case

Open the case and remove the lid. The insoles won't charge if the lid is closed. Take out the power cable from the case and connect it to the case, close to the On/Off switch. Plug the other end to a power outlet. Switch it on (position I). If the charging slots are empty, the LED will flash white for a few seconds then go off.

The charging case should be at a temperature below 35° Celsius.

### Charging the insoles

To position a pair of insoles in a charging slot, identify the insole size and locate the corresponding slot. Position each insole on the edge, top side facing outwards.

## c. Charge light indicators

- Continuous green : charged
- Continuous white : charging
- Discontinuous white : Insoles poorly positioned, reposition
- Light off : power cable disconnected or improper positioning of the charger or the insole. Check position to make the light indicator steady white.

When the insoles are charging, the indicator light of the chargers is solid white. Once the insoles are charged, the indicator turns green. The indicator is flashing white when an insole is incorrectly positioned, the insole is not charging. It takes approximately 2.5 hours for a complete charge of the insoles.

## d. Bluetooth connection to the insoles

Once charged, the insoles are automatically detectable via Bluetooth using FeetMe® Evaluation app. The insoles go to sleep mode automatically when not in use. In order to scan them, just shake them for 3 seconds. No manipulation is therefore necessary from the user part, to turn the insoles on and off. If the insoles are not detected, consider charging them. Be careful not to use the insoles alongside other Bluetooth devices.

## 10. Use of insoles with FeetMe® Evaluation mobile application

### a. First use or reinstallation

Create a Gmail account if you do not have one. Go to the « Play Store » and search for the application : « Feetme Evaluation ».

Download the application and follow the installation instructions.

### b. Perform a gait recording

#### 1. Connection to the insoles

Click on the button « Connect the insoles », select the right and left soles of the corresponding size and then click on « Next step ».

#### 2. Insoles calibration

For this step, the insoles must be in the shoes, laid on a flat surface. Click on the « Calibration » button.

#### 3. Select a test

Click on the chosen test.

You can also acquire gait parameters without the phone and synchronize the data at the end of the recording by clicking on « Import Recording ».

#### 4. Start & stop recording

Put on shoes and press « Start » then « Play » to start recording. For « Standard » tests, press « Stop » and « Save » to save the data. For other tests, stopping the recording and saving the data are automatic.

#### 5. Make a new recording

You remain connected by default to the last insoles, so you can make other recordings without redoing the steps of connection and calibration (repeat steps from step 3). To make a new recording with other insoles, click on « Connect the insoles » and repeat the above steps from the beginning.

### c. View the data acquired from the mobile application

The data collected for a given patient can be viewed from the application. To access it, go to the « Recordings » tab to access all the records. To access the data of a record, click on the associated record.

### d. Data backup

In case the user hosts the gait recordings, he is in charge of performing appropriate and regular data backups in order to prevent any destruction or loss of information. FeetMe cannot be held responsible for any accidental or unauthorised destruction or accidental loss of user data.

### e. View battery level

The battery level of the insoles is visible on the insoles selection page and also, once the insoles are connected, on the recording screen.

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It is also possible to configure the automatic update installation.

### 11. Troubleshooting and deactivation

Defect	Cause	Action to be taken
Insoles undetected	Uncharged insole	Fully charge the insoles.
	Inactive Bluetooth	Activate the Bluetooth of your smartphone.
	Electromagnetic noise	Do not wear the insoles on an electromagnetically noisy place.

Defect	Cause	Action to be taken
Insole is undetected	Error on the insoles	 Reset the insole press the « reset » button (with the tip of a pen for example) for 10 seconds. The reset button is located on the top of the insole, near the logo.
Insole is undetected	Phone's Bluetooth mode does not work	Disable the Bluetooth mode on your phone and then turn the Bluetooth mode back on.
None	N/A	For the deactivation of the insoles, the user must disconnect the charging cable and the charger's adapter, remove the insoles and put them in a place that is free of mechanical and electrical stress.

### 12. Storage and transport

It is recommended to transport and store FeetMe® Monitor insoles under the following conditions :

- Ambient temperature between -25 °C and + 70 °C.
- Maximum humidity of 93%.
- Pressure between 700 hPa and 1060 hPa.

#### Transport

If you are transporting FeetMe® Monitor insoles without wearing them, it is advisable to put them in their packaging. For air transport, it is advisable to put the insoles in your luggage in the hold.

It is advisable to avoid dropping off the device. A fall can alter the performance of the product.

#### Storage

#### Warning

In case of non-use of your device, it is advisable to keep it away from dust and in a clean and dry place. FeetMe® insoles should not be stored near sources of heat or fire.

### 13. Cleaning and maintenance

It is recommended to perform the following actions regularly :

1. Clean FeetMe® insoles with a damp cloth or wipe or with an antifungal product.
2. Let them air dry or wipe with a clean, dry, lint-free cloth.

#### Warning

Be careful, do not use corrosive products, oil, acetone or petroleum products to clean FeetMe® insoles. Do not clean FeetMe® insoles in a dishwasher, a tumble dryer or other steam appliance. Do not immerse FeetMe® insoles.

### 14. Disposal

The device contains electronic components. It is forbidden to dispose off FeetMe® insoles along with household waste. The device must be brought back to your healthcare provider or to its manufacturer.

## 15. Technical characteristics

Use condition of the insoles	Working temperature	+5° to +40°C
	Storage and transport temperature	-25° to +70°C
	Humidity for use, storage and transport	15 % to 93 %
	Altitude range	0-3000 m
	Pressure range	700 hPa - 1060 hPa
	Maximum charge	135 kg
	Maximum measured pressure	15 kg/cm <sup>2</sup>
	Power supply	3,7 V DC, 110mAh
Protection index	IP55	IP5X: Protected against dust (limited ingress, no harmful deposits). IPX5: Protected against low pressure water jets from any direction. Limited ingress permitted.
Measured parameters	Plantar pressure Stride length Walking speed	In kg/cm <sup>2</sup> +/- 2,5kg/cm <sup>2</sup> In meters +/- 0,1m In meters/ seconde +/- 0,1m/s

Insoles	Material	Polyamide, Thermoplastic Elastomer (TPE), Fabric
	Dimensions	Thickness : 4.5+/- 0.1mm
	Weight	Maximum 100g
Wall adapter	Manufacturer : Globtek, Inc - Ref : GTM41078-0605-USB 5 V DC - Norme IEC 60601-1	
Lithium ion type battery	Life time	1 year
	Certification	IEC 62133-2: 2017
Double head charging cable	Induction charging	5V- 1ampere
Charging case	Power supply	Input: 230AV 50Hz 138W Output: 5VDC / 17A
	Operating temperature	+5° to +35°C
	Storage and transport temperature	-25° to +70°C
	Dimensions (cm)	50 x 23 x 70
	Weight	13kg
	Material	EVA, MDF

## 16. Recommended minimum phone features

Designation	Value
OS	Android
Version	5.1.1
Bluetooth v4.0	4.0

Be sure to maintain a safe distance between FeetMe® Monitor and any high-frequency emitting device to prevent the possibility of malfunction.

This equipment complies with the IEC 60601-1-2: 2015 norm for the electromagnetic compatibility of the equipment and/or medical electrical systems.

## 17. Characteristics of the radio module

Module	Characteristics	Value
Bluetooth v4.2 low energy	Emitted power	+4 dBm
	Frequency	2400-2483,5 Mhz

Portable and mobile RF communication devices may compromise the operation of other medical electrical devices.

This standard is intended to provide a reasonable protection against harmful interference in a typical medical installation. However, given the rise of radio frequency wave along with other parasite sources in health care and other environments, it is possible that high levels of interference due to close proximity or due to the power of a source, disrupts the operation of this device. Medical electrical devices require special precautions regarding electromagnetic compatibility and all devices must be installed and put into service in accordance with the information specified in this manual.

NOTE 1 : At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 : These guidelines may not apply to all situations. Electromagnetic propagation is affected by the absorption and by the reflection of structures, objects and people.

- › The field strengths of fixed transmitters, such as base stations for radiotelephones (mobile/wireless) and land mobile radios, amateur radio, AM and FM broadcasting, and TV broadcasting, cannot be to be theoretically predicted accurately.
- › In order to evaluate the electromagnetic environment due to fixed RF transmitters, an on-site electromagnetic investigation should be considered. If the field strength, measured at the location where the insoles are used, exceeds the applicable RF compliance level above, the insoles should be observed to verify that operation is normal. If abnormal performance is observed, additional measures may be necessary, such as reorienting or repositioning the insoles.
  - › In the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V m.
    - › Conformity
    - › Feetme® Monitor complies with IEC 60601-1-2:2014 (Ed.4) and EN 60601-1-2:2015

## 18. Maintenance

The maintenance of the insoles must not be carried out by a third person. In case of a malfunction, please contact your distributor or the manufacturer.

## 19. Lifetime

The lifetime of the device is the smallest duration between two years or one million steps.

This duration does not apply to consumable parts, such as batteries or protective materials, which are expected to wear out over time, unless the damage is due to a defect in materials or manufacturing.

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